


RUDOLF

BECAUSE WE CARE



Gynaecology extension set

Laparoscopic sacrocolpopexy

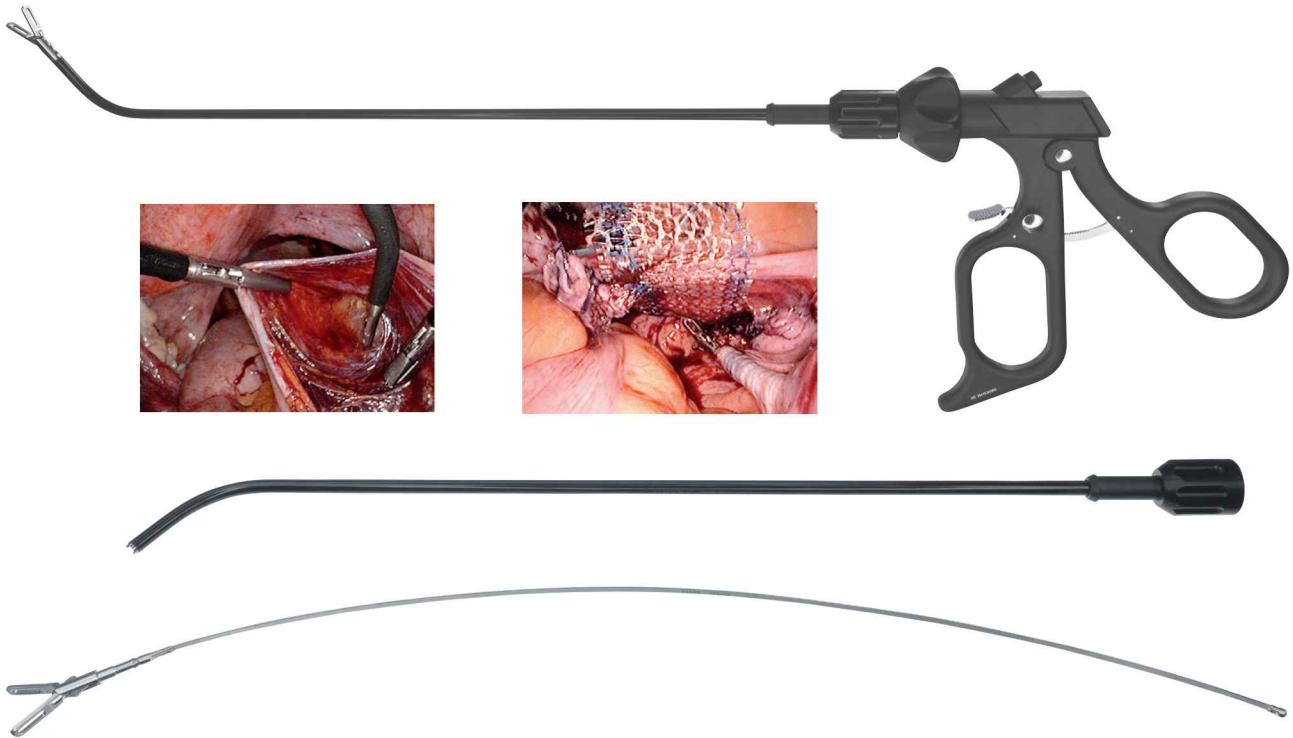

Description	Ordering no.
Grasping forceps, completely dismountable, curved, Ø 5 mm	RS818-864
Handle with ratchet, without HF connection	RS810-000
Curved guiding tube	RS008-000
Insert grasping forceps, fenestrated	RS000-864



Description	Ordering no.
Flexible trocar, 6 mm, complete, 70 mm Obturator	RU 0430-06
Scissors for delicate tissue reconstruction, 18 cm	RU 1321-18M

Piraña Cut - One instrument three functions



Due to its several functions the Piraña Cut reduces the necessity of instrument exchanges during an operation and thus helps minimize the loss of CO₂.




Scissors

Sharp scissors for the final cut after the suturing procedure.
Recommended suture material:
monfil suture 0-2/0
polyfil suture 0-2/0 absorbable.



Forceps

Teeth 1:2 enable a tight grip of the tissue prior to suturing.

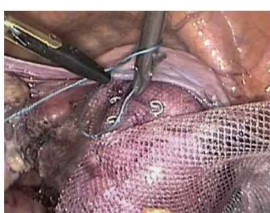

Needle Holder

The fine structure of the jaw functions as a second needle holder thus assisting in the taking/handling over of the needle from/to the laparoscopic needle holder.




Special Coating

The black coating of the insert allows for a longer lifetime of the scissors blades and causes less reflexion.




A complete **SCREW-IN** Instrument consisting of:
- Handle, Tube, Insert



		Multifunctional laparoscopic instrument Piraña Cut, 1:2 teeth, double action, Ø 5 mm		
Insert, only	Length	Ordering no.		
RS000-595	330 mm	RS115-595	RS135-595	RS225-595
RS000-795	450 mm	RS117-795	RS137-795	RS227-795



RUDOLF Medical GmbH + Co. KG, represented worldwide, translates its belief in lasting values by fostering close customer relationships and offering innovative high technology products, hand instruments, rigid endoscopes as well as efficient services for numerous surgical disciplines.

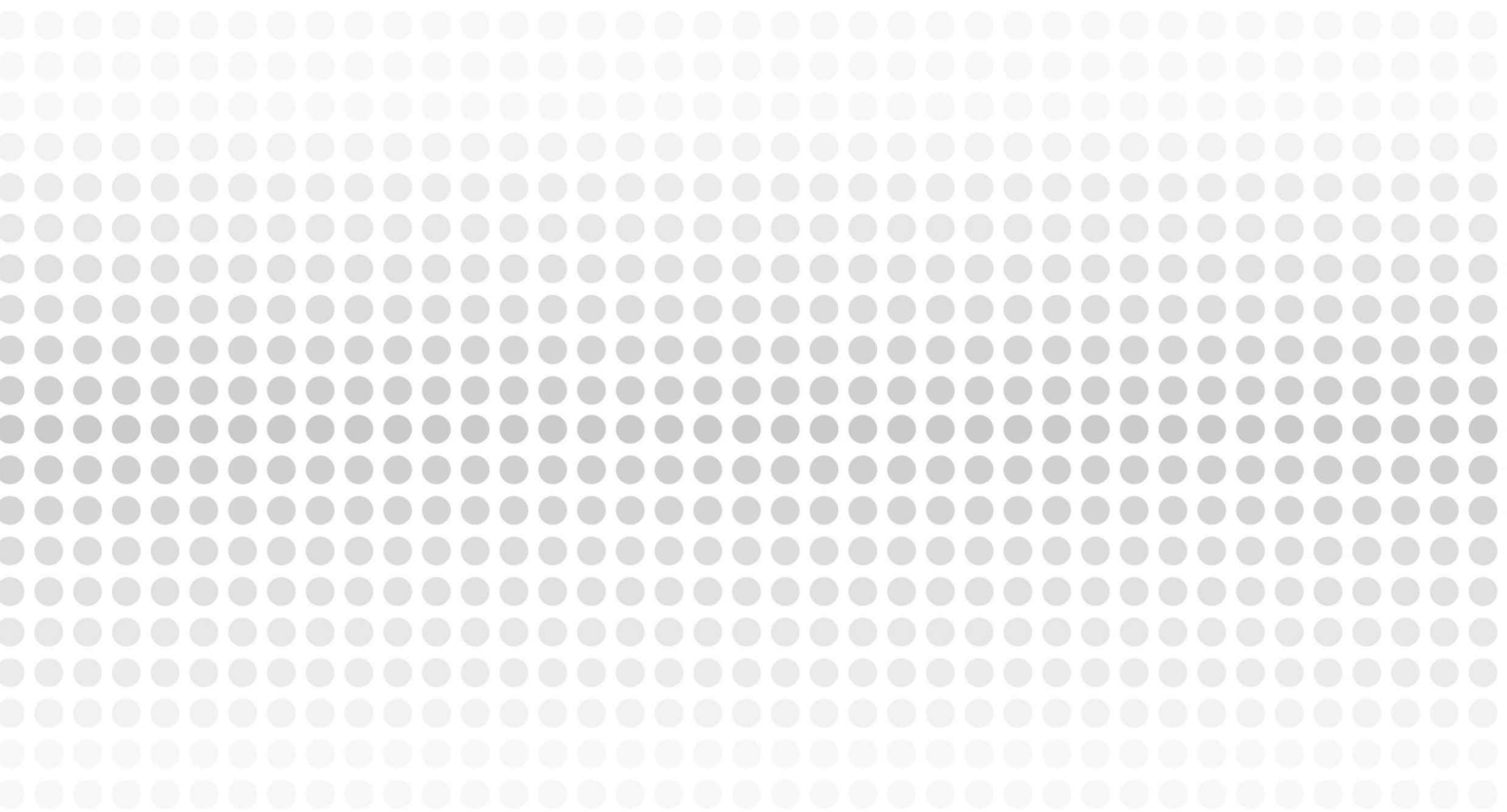

**MITTELSTAND
GLOBAL**
HEALTH MADE
IN GERMANY

RUDOLF Medical GmbH + Co. KG
Zollerstr. 1, 78567 Fridingen, Germany
Tel. +49 7463 9956-0
sales@rudolf-med.com
www.rudolf-med.com
BECAUSE WE CARE



ZK 576 EN_Rev. 12.2017

Distributed by:

